

Cuspidor and Assistant Arm Preventative Maintenance:

Daily Maintenance:

If you are using surface disinfectants, then it is very important to remove the residue they leave behind daily.

Never spray disinfectant directly onto the surface of the equipment.

For plastic and metal parts, remove ordinary dirt and smudges with a mild soap and water solution and a clean, soft cloth or towel; dry with a soft, lint-free cloth or towel.

For the Cuspidor:

Flush the Cuspidor drain;

Pour 1 quart of disinfectant (10 parts hot water and 1 part chlorine bleach) into the cuspidor bowl and raise the chair to the full up position.

For the Assistant Arm:

Clean the HVE and Saliva Ejectors with an evacuation line cleaner. (follow the manufacturer's line cleaning protocol)

Weekly maintenance:

For the HVE and Saliva Ejector:

Remove the HVE or Saliva Ejector valve from the tubing, separate the valve handle from the body and thoroughly clean any debris or build up.

Reassemble the valve and disinfect.