

Instructions for use 1407

1. Turn the Master On/Off toggle to the 'On' position. It is located under the left side edge of the control head.
2. The handpieces are automatically activated when removed from their hanger.
3. Depress the Foot Control to operate the activated handpiece. Pressure is shown on the gauge for the handpiece being used.

- **NOTE:** Individual handpiece pressure adjustments are located inside the control head.

To adjust the pressure to each handpiece, turn the adjustment screw clockwise to decrease pressure or counterclockwise to increase pressure.

CAUTION: When adjusting the handpiece pressure, do not over-tighten the screws. This may result in unnecessary damage to the handpiece control block.

4. To activate water spray, turn the Water On/Off Toggle to the 'On' position. It is located on the delivery system foot control. Flow adjustment is accomplished by adjusting the flow control knobs under each hand piece position.
5. To adjust the air water mix adjust the "coolant air" flow control knob located on the far right-hand side of the unit next to the water flow knobs.
6. To release the air lock of the unit arm, activate the momentary push button valve located on the handle of the delivery system, adjust the the height of the flex arm to desired position. When desired position is achieved, release the momentary button switch to lock the height of the flex arm.

CAUTION: Do not attempt to adjust the height of the flex are without releasing the air-brake valve. Failure to release the air-brake valve may cause damage.