

## 6000EL QUICK INSTALLATION GUIDE (NO ARMREST)


1

Remove box from pallet.




2

Using a ½" wrench, remove the shipping bolts and place the chair in its final location. You may now connect the chair to a power source.

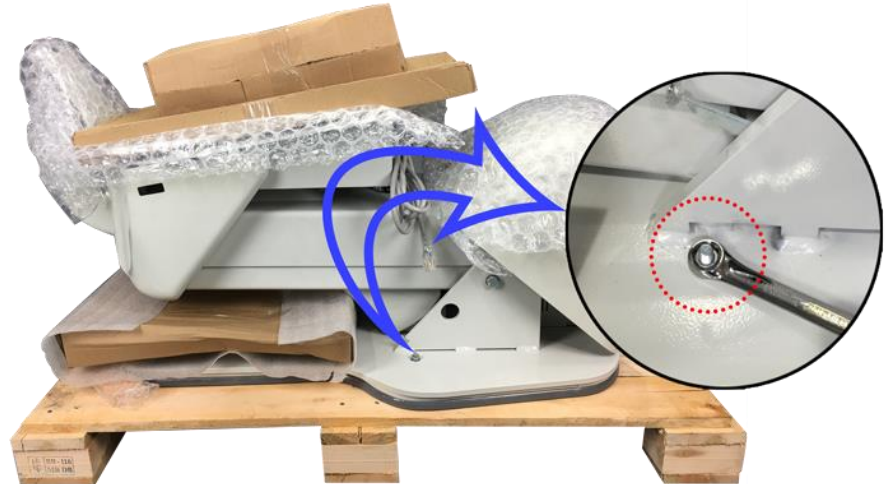


3

Lower backrest articulation bracket (1), and install the backrest frame (2) using four ¼"-20 bolts (3). NOTE: Use the chair functions to adjust the chair position to ease installation.

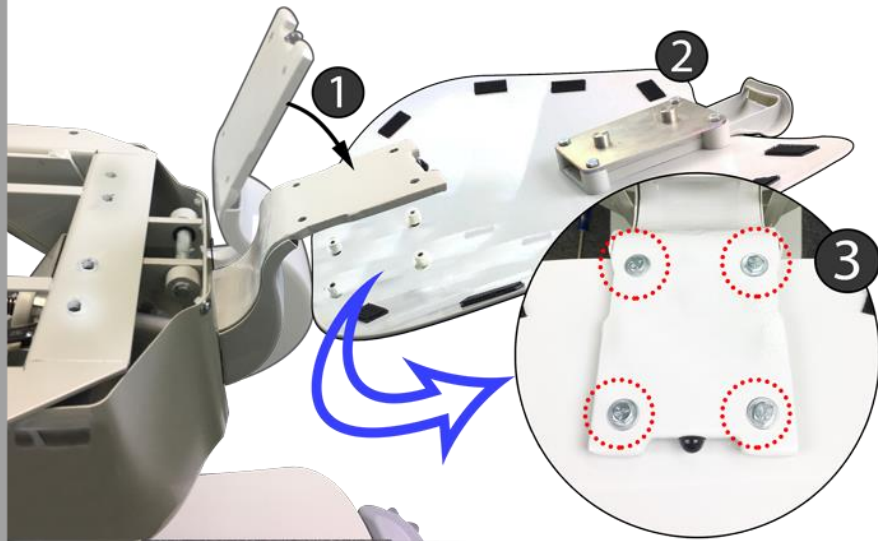

4

Install toe pan using two sets of 3/8" bolts, nuts and washers.

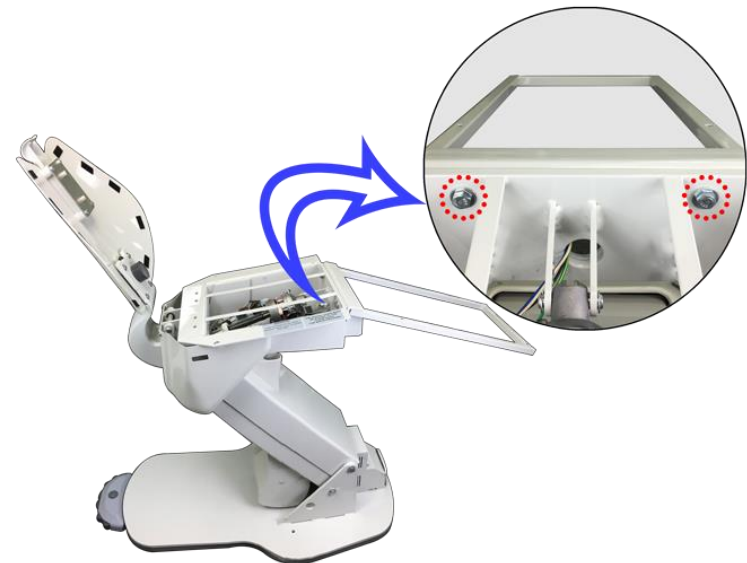

# 6000EL QUICK INSTALLATION GUIDE (NO ARMREST)


5

Install cover using three Philips screws.



6

Put the headrest in position as shown in the image.

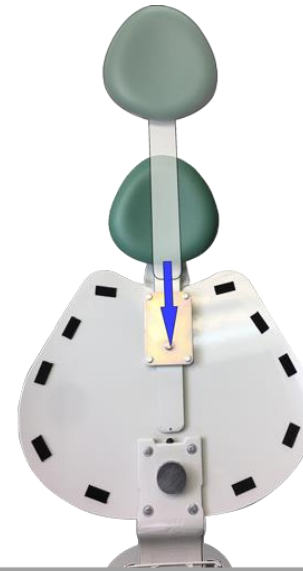

7

Lower backrest, align back cushion on frame, apply pressure on areas indicated until Velcro locks in place.

