


7000BY QUICK INSTALLATION GUIDE


1

After removing the box from the pallet. Remove the pump cover via 3 Philips screws.

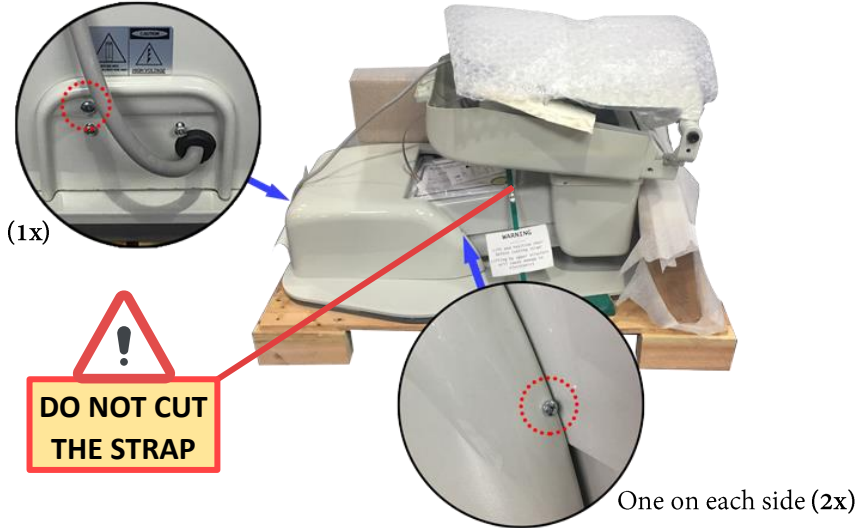


2

Using a ½” wrench, remove shipping bolts. Remove shipping blocks.

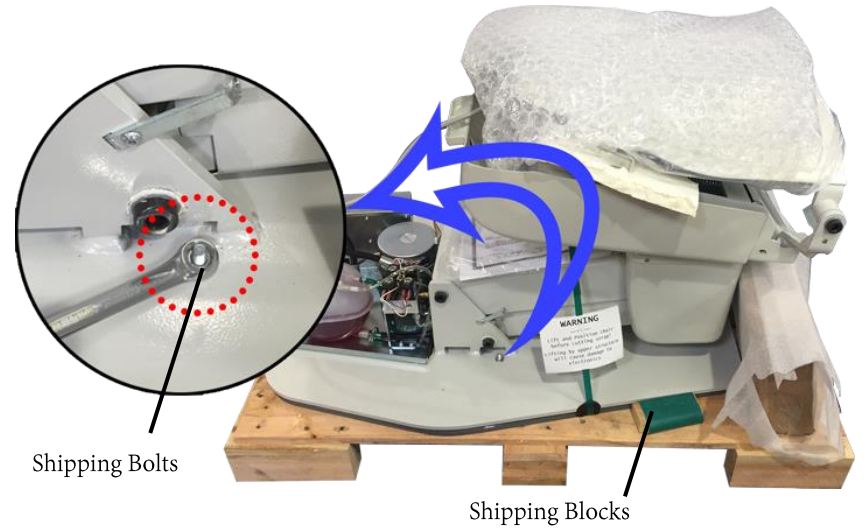


3

Lift the chair off the pallet using the backrest/armrest mounting brackets.

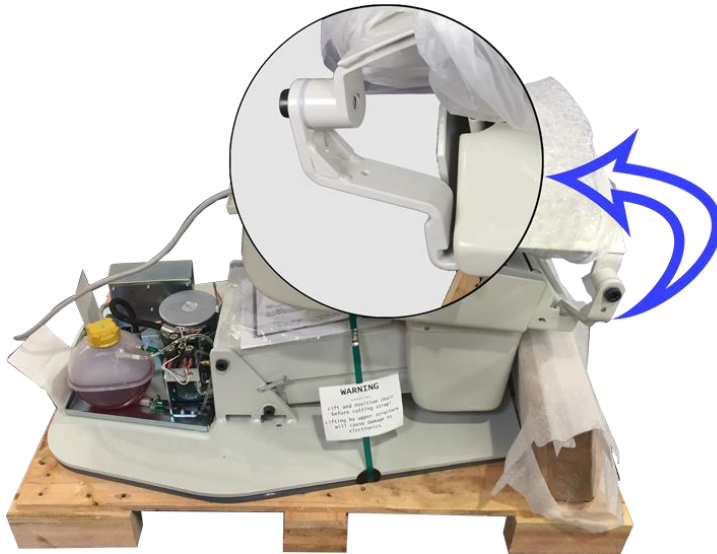


4

Once the chair is positioned in its final location, cut the strap and remove the shipping screw from the reservoir cap.

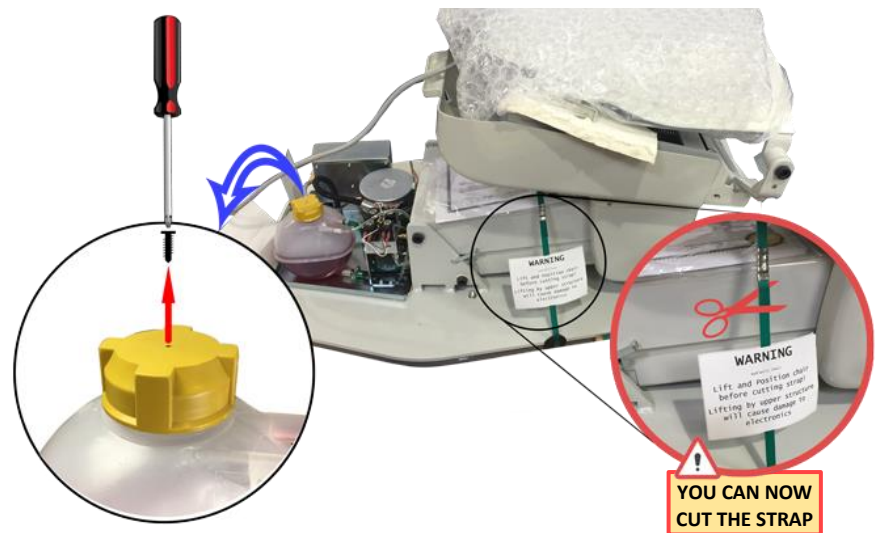

7000BY QUICK INSTALLATION GUIDE


- 5** Raise base to full up position, lift backrest, install pin, and tighten set screw.

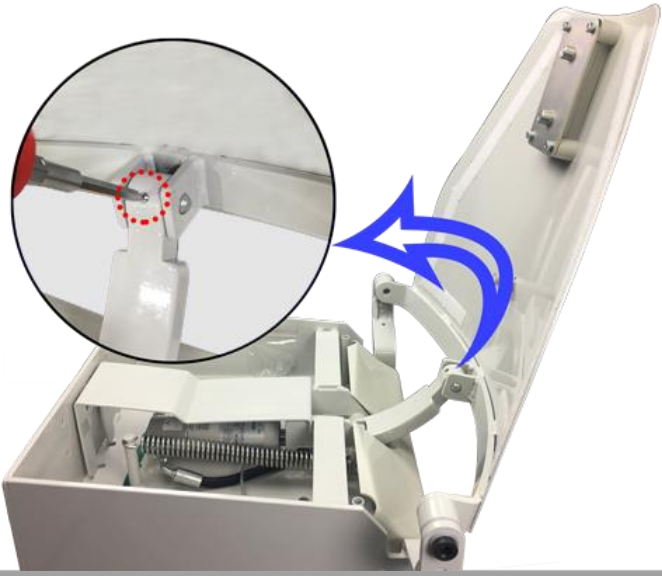

- 6** Remove back piston cover.



- 7** Install toe pan and armrest.

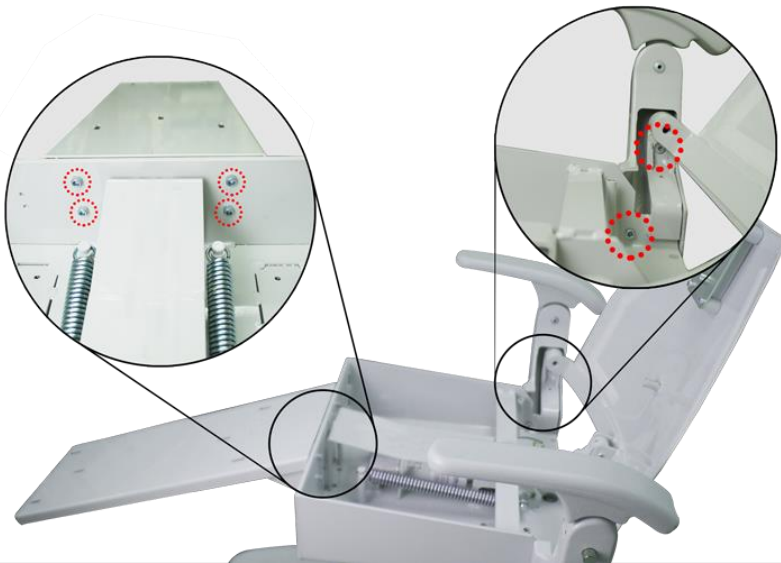

- 8** Install upholstery, re-install all covers and test all chair functions.

