


7000BY QUICK INSTALLATION GUIDE (NO ARMREST)


- 1** After removing the box from the pallet. Remove the pump cover via 3 Philips screws.




- 2** Using a ½” wrench, remove shipping bolts. Remove shipping blocks.

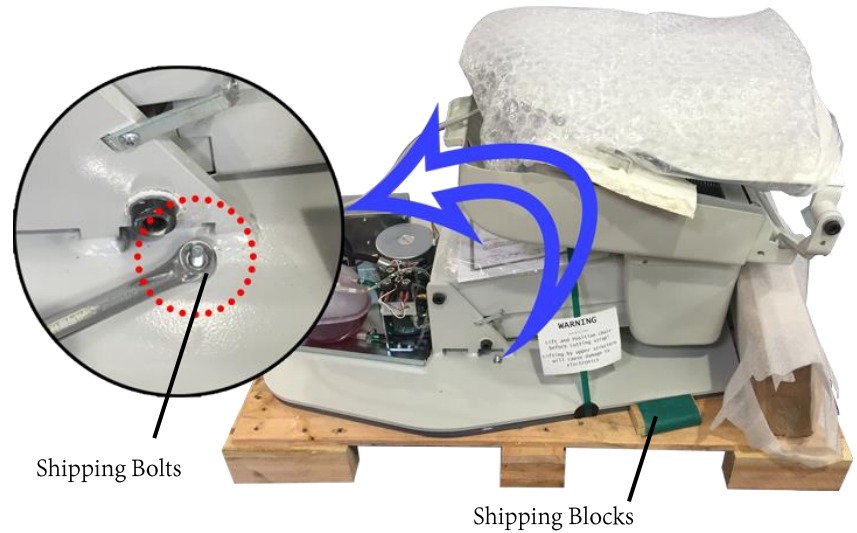

- 3** Lift the chair off the pallet using the backrest/armrest mounting brackets.

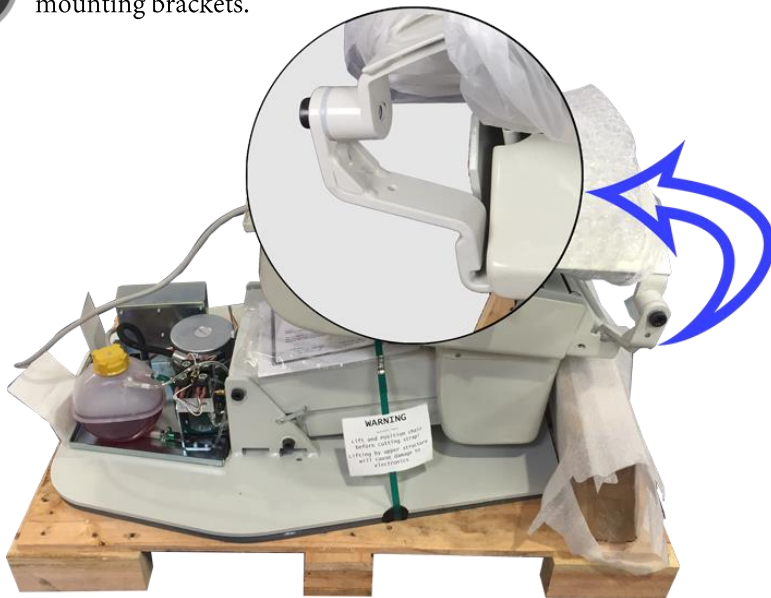


- 4** Once the chair is positioned in its final location, cut the strap and remove the shipping screw from the reservoir cap.

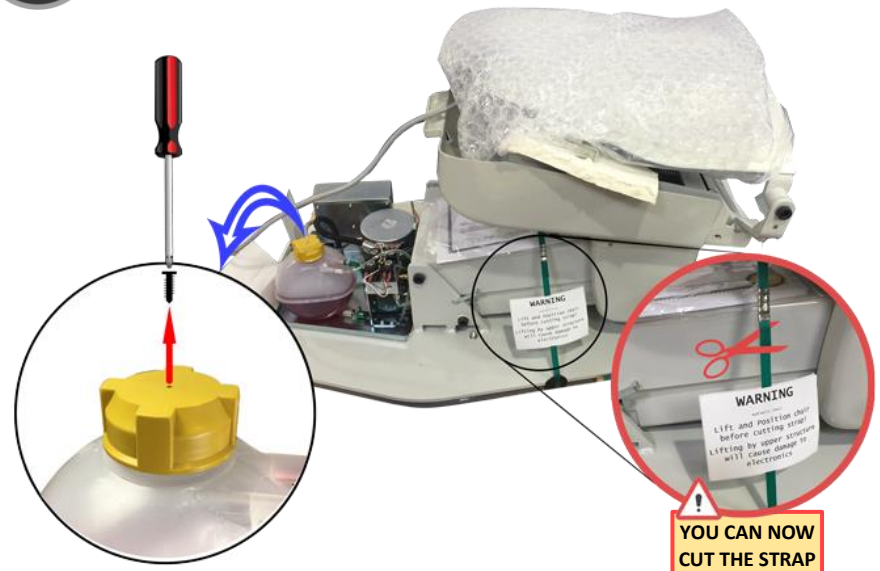

7000BY QUICK INSTALLATION GUIDE (NO ARMREST)

5

Raise base to full up position, lift backrest, install pin, and tighten set screw.

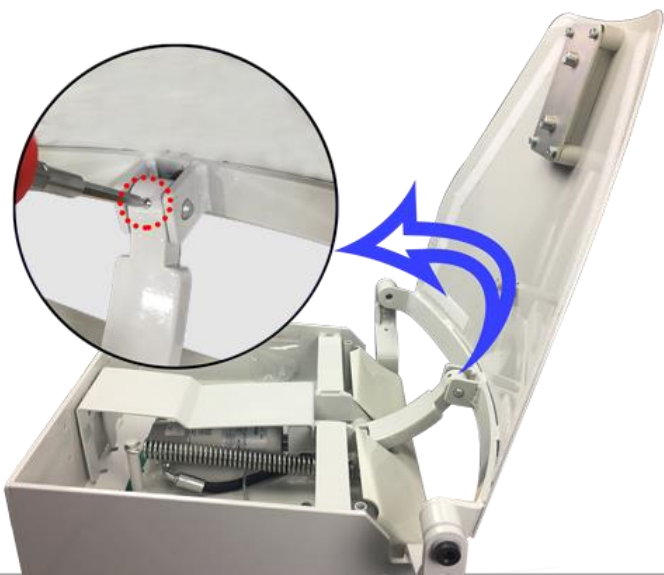

6

Install the toe pan using a 1/4" Allen wrench.



7

Install upholstery, re-install all covers, and test all chair functions.

