



9000PB QUICK INSTALLATION GUIDE


- 1** After removing the box from the pallet. Remove the pump cover via 3 Philips screws.

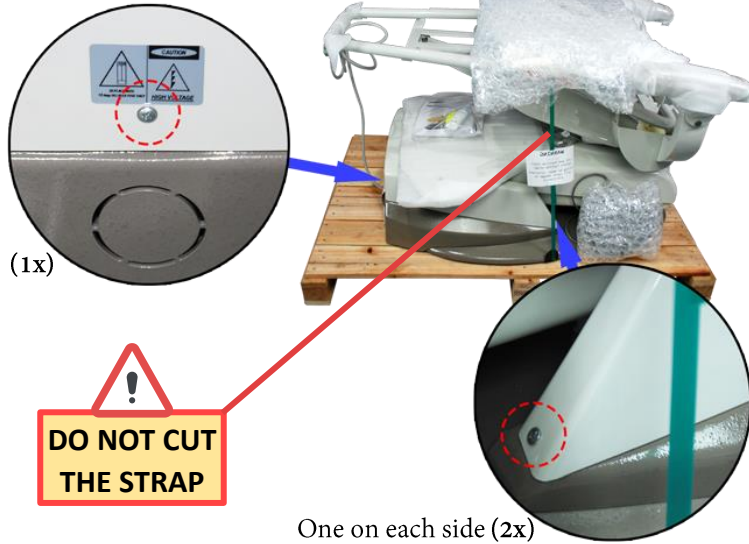


- 2** Using a ½” wrench, remove shipping bolts. Remove shipping blocks.

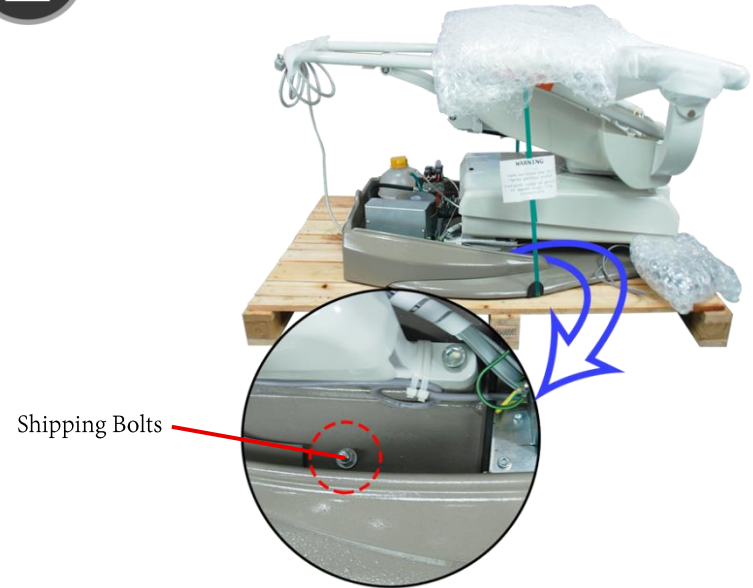

- 3** Lift the chair off the pallet using the armrest mounting brackets.

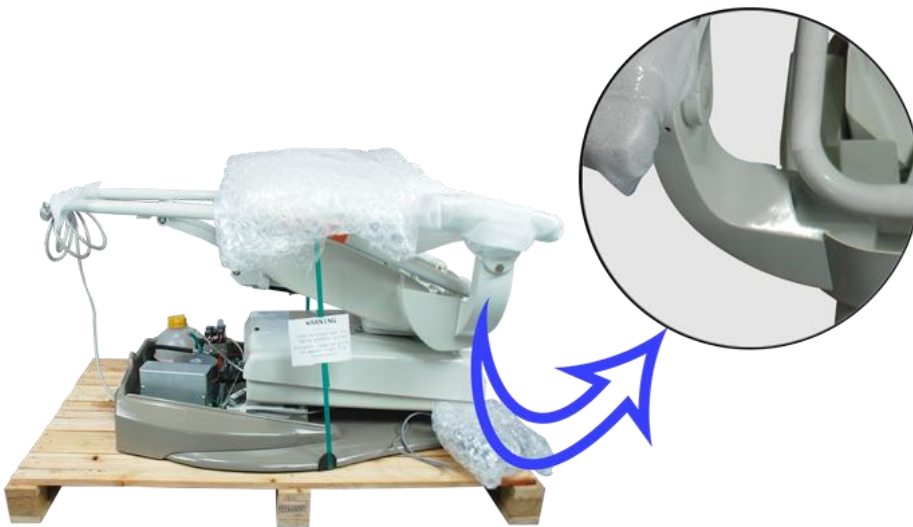


- 4** Once the chair is positioned in its final location, cut the strap and remove the shipping screw from the reservoir cap.

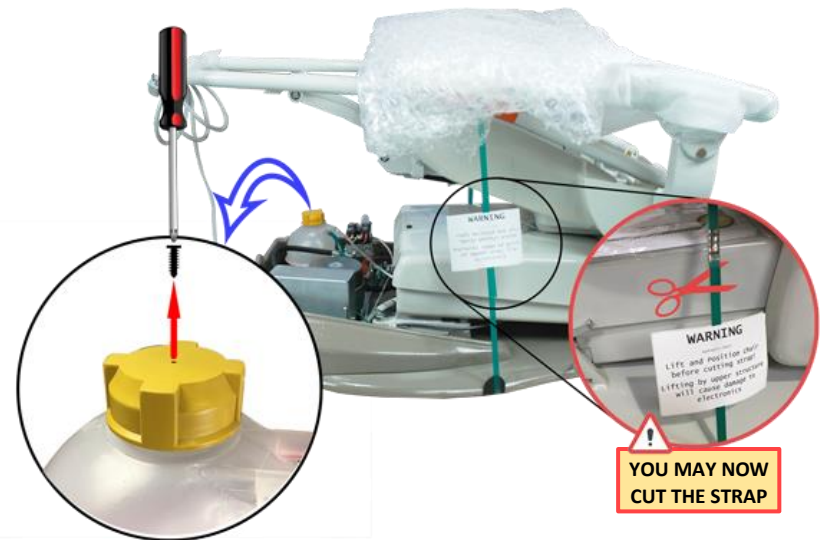

9000PB QUICK INSTALLATION GUIDE


5

Remove one E-clip from one side of the pin and remove the pin. Align backrest link with the backrest bracket, reinsert the pin and secure it with E-clip.

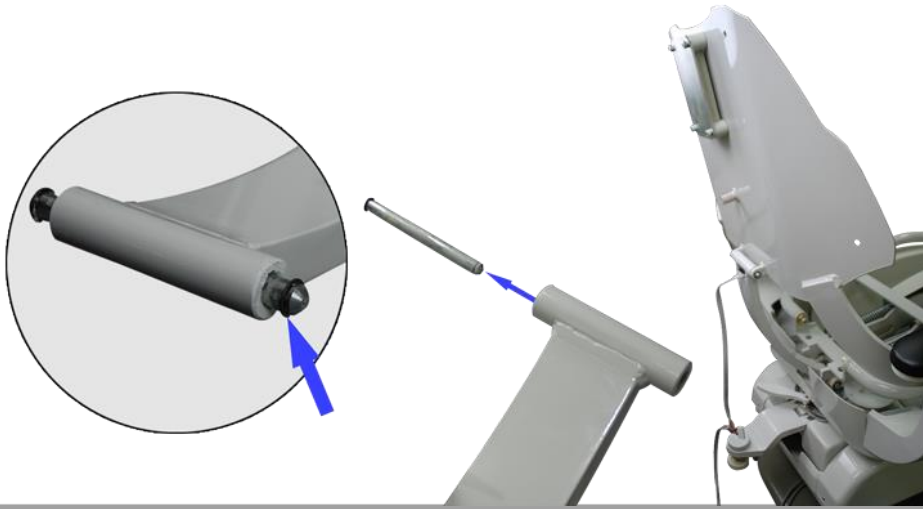


6

Align the backrest cushion threaded nuts with the cushion holes in the backrest frame, press firmly to lock the cushion in place. While holding the backrest cover connect the membrane switches. Align terminal pin locking tangs on both connectors.

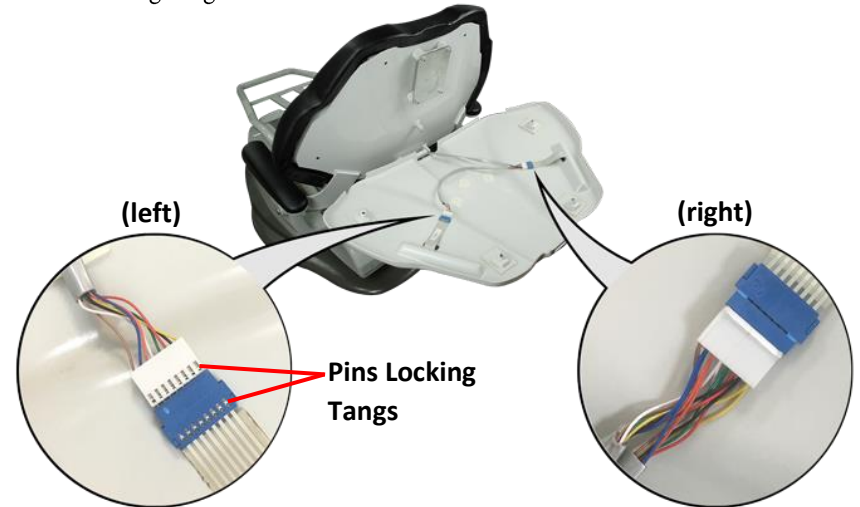

7

Before installing the backrest cover, test both membrane buttons. Install backrest cover using four screws. Insert caps to cover the screw holes

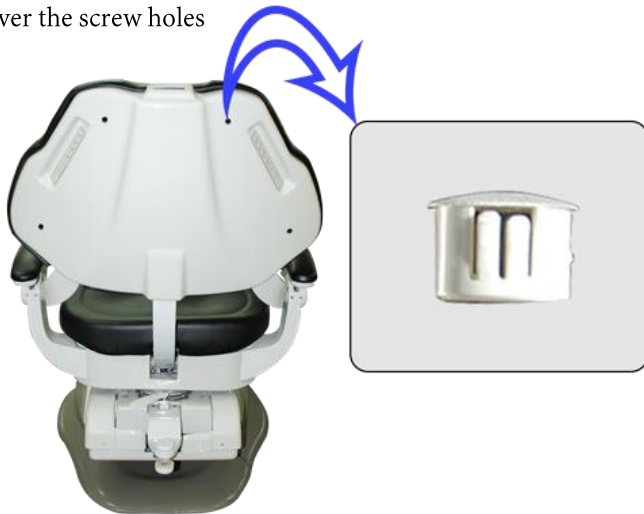

8

Install headrest, upholstery, re-install all covers, and test all chair functions.

