

5000FB, 6000EL, 6700M Preventative Maintenance

Daily Maintenance:

If you are using surface disinfectants, then it is very important to remove the residue they leave behind daily.

For Upholstery, plastic and metal parts, remove ordinary dirt and smudges with a mild soap and water solution and a clean, soft cloth or towel; dry with a soft, lint-free cloth or towel. The use of vinyl “conditioners” or “protectants” is not recommended and should be avoided on all upholstery.

Annual Maintenance:

Check chair fasteners (bolts, screws, set screws, nuts, etc.) for tool tightness.

Lubricate Pivot points and bushings;

1. Remove arm covers (if applicable), trip pan/safety plate cover, seat upholstery, and motor cover.
2. Lubricate all pivot points and motor shafts.

