

## Cleaning and Disinfection

### Day-to-day soil:

Remove ordinary dirt and smudges with a mild soap and water solution and a clean, soft cloth or towel; dry with a soft, lint-free cloth or towel. The use of vinyl “conditioners” or “protectants” is not recommended and should be avoided on all upholstery.

### Barrier Technique:

The first choice in the protection of dental equipment should be the use of disposable barrier products. The repeated use of disinfectant on equipment surfaces without the constant removal of the solution residue will eventually cause some damage to equipment surfaces.

### Chemical Disinfecting:

For upholstery; Never use spray disinfectants, always use wipes. Use a non-alcohol, non-chorine, based wipe for plastic, metal, and upholstered surfaces.

**Always clean the residue with mild soap and water.**

**CAUTION:** Should you desire to use other cleaning methods, carefully try them in an inconspicuous area to determine potential damage to the material. Never use harsh solvents or cleaners which are intended for industrial applications. Avoid use of paper towels.

**CAUTION:** Cleaning products may be harmful/irritating to your skin, eyes, etc. Use protective gloves and eye protection. Do not inhale or swallow any cleaning product. Protect surrounding area/clothing from exposure. Use in a well-ventilated area. Follow all product manufacturer’s warnings. Summit Dental Systems cannot be held responsible for damage or injuries resulting from the use or misuse of cleaning products.

The Summit Dental Systems warranty does not cover damage to equipment and upholstery caused by cleaning and disinfectant solutions.