


6700M QUICK INSTALLATION GUIDE


1

After removing the box from the pallet. Remove the pump cover via 3 Philips screws.

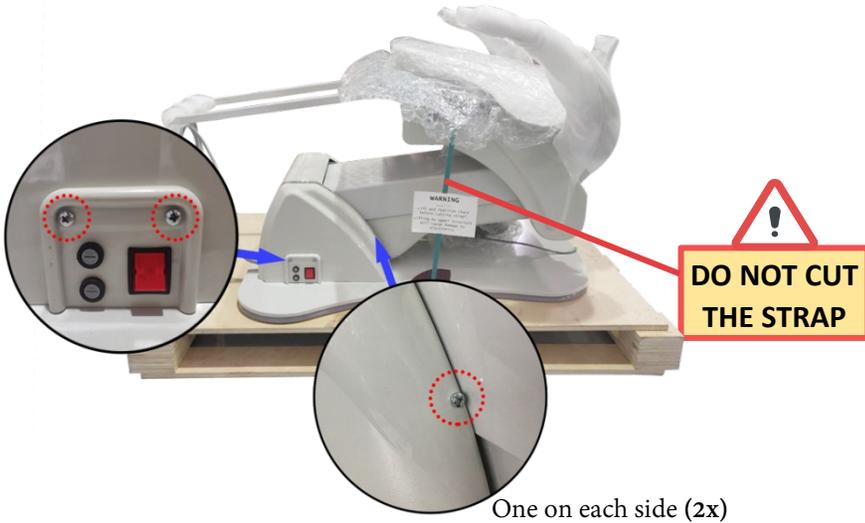


2

Using a ½” wrench, remove the shipping bolts.

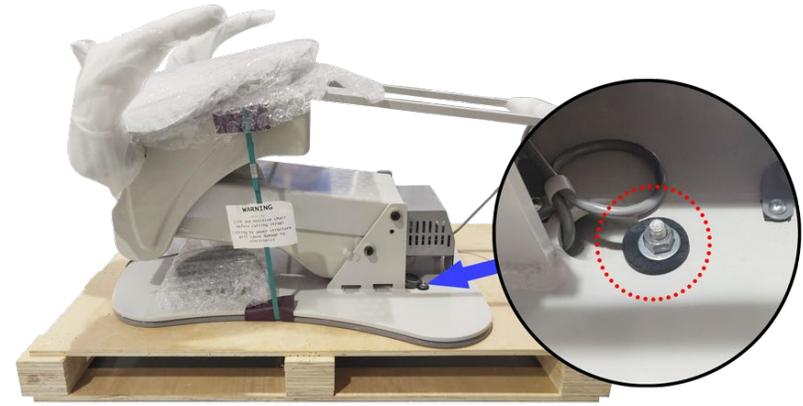

3

Lift the chair off the pallet using the backrest/armrest mounting brackets. NOTE: Do not lift the chair from the toe pan bracket.

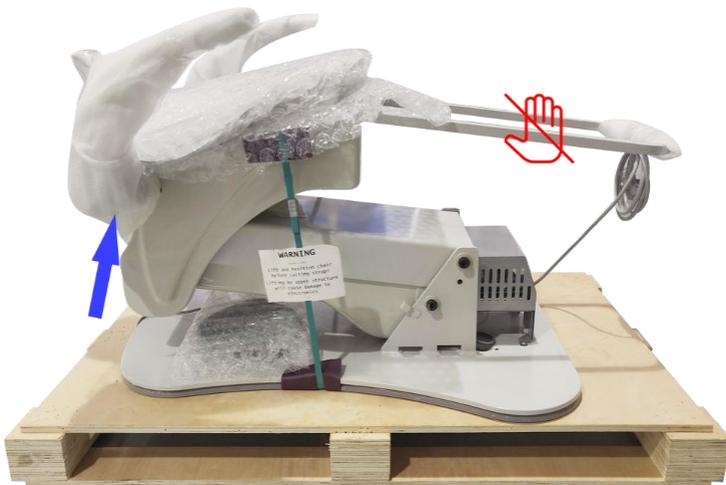

4


Once the chair is positioned in its final location, you may now cut the shipping strap



6700M QUICK INSTALLATION GUIDE


5

Lower backrest articulation bracket, and install the backrest frame using four ¼" bolts. NOTE: Use the chair functions to adjust the chair position to ease installation.

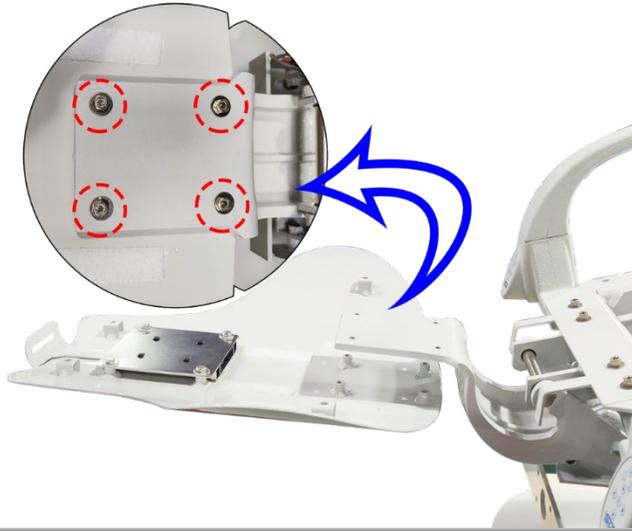


6

Put the headrest in position as shown in the image.

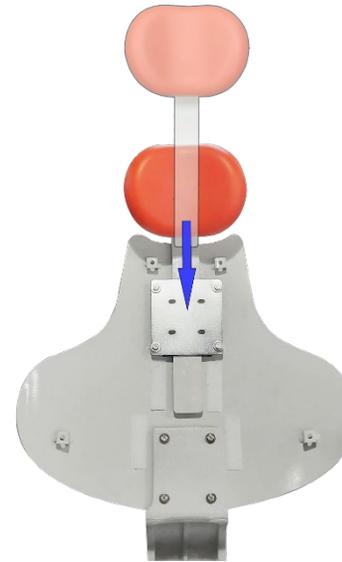

7

Install the cover back.

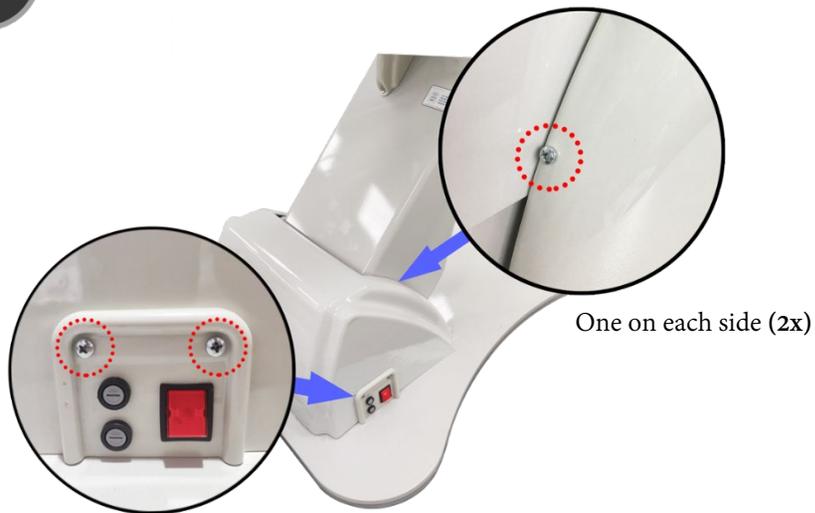

8

Install upholstery, re-install all covers and test all chair functions.

