

7000BY, 8000DY, 9000PB Preventative Maintenance

Daily Maintenance:

If you are using surface disinfectants, then it is very important to remove the residue they leave behind daily

For Upholstery and plastic and metal parts, remove ordinary dirt and smudges with a mild soap and water solution and a clean, soft cloth or towel; dry with a soft, lint-free cloth or towel. The use of vinyl “conditioners” or “protectants” is not recommended and should be avoided on all upholstery.

Annual Maintenance:

Check Fluid Level in reservoir with the back and base in the full down position.

Check chair fasteners (bolts, screws, set screws, nuts, etc.) for tool tightness.

Lubricate Pivot points and bushings;

1. Remove arm covers (if applicable), trip pan/safety plate cover, seat upholstery, and pump cover.
2. Lubricate all pivot points and piston shafts.

