


## Instructions for use 1402

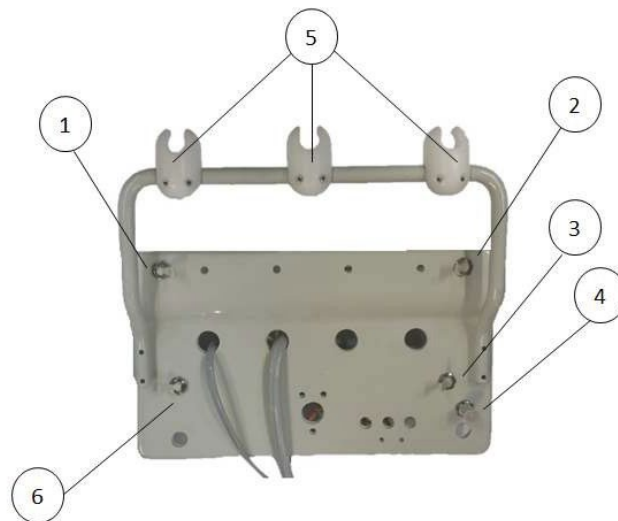

1. Turn the Master On/Off toggle to the 'On' position. It is located under the left side edge of the control head.
  2. The handpieces are automatically activated when removed from their hanger.
  3. Depress the Foot Control to operate the activated handpiece. Pressure is shown on the gauge for the handpiece being used.
    - **NOTE:** Individual handpiece pressure adjustments are located at the bottom rear of the control head.  
To adjust the pressure to each handpiece, turn the adjustment screw clockwise to decrease pressure or counterclockwise to increase pressure.
- CAUTION:** When adjusting the handpiece pressure, do not over-tighten the screws. This may result in unnecessary damage to the handpiece control block.
4. To activate water spray, turn the Water On/Off Toggle to the 'On' position. It is located on the right-side edge under the control head. Flow adjustment to the 'wet' handpieces is controlled by the controls labeled 'Flow Control'.

5. To release the air lock of the unit arm, activate the momentary toggle valve and adjust the height of the flex arm to desired position. When desired position is achieved, release the momentary toggle switch to lock the height of the flex arm.

**CAUTION:** Do not attempt to adjust the height of the flex are without releasing the air-brake valve. Failure to release the air-brake valve may cause damage.