


# 8700DY CHAIR - QUICK INSTALLATION GUIDE


1

After removing the box from the pallet. Remove the pump cover via 2 Philips screws.

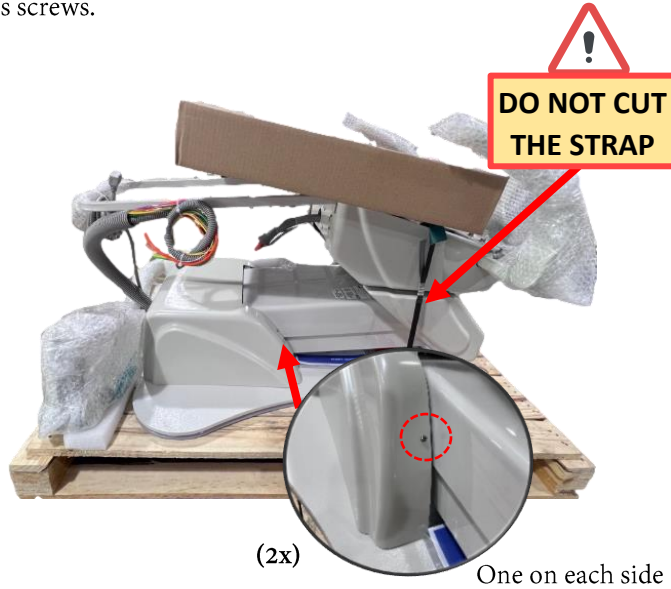

2

Using a ½" wrench, remove shipping bolts. Remove shipping blocks. Do NOT cut the strap yet.

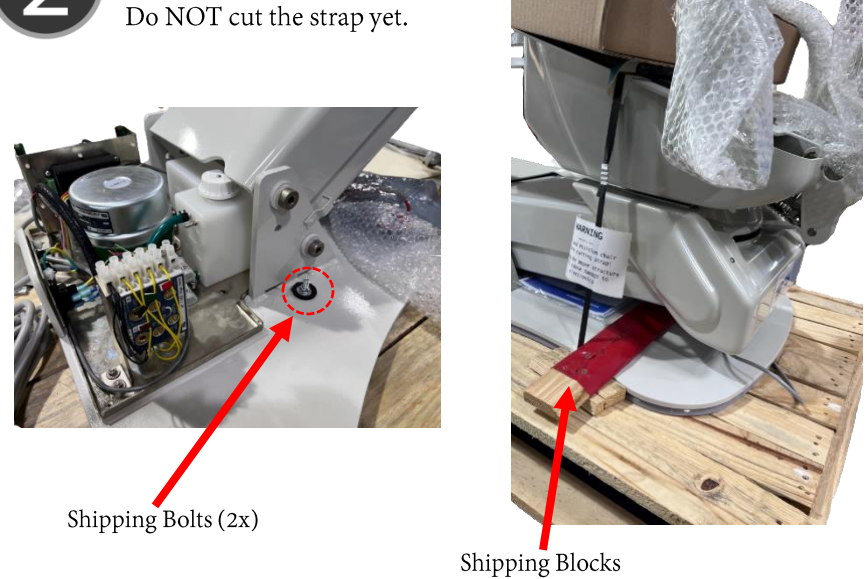

3

Do NOT lift the chair off the pallet, but slide/push using the bottom of the backrest/armrests mounting brackets.

NOTE: Do not lift the chair from the toe pan bracket.




4

Once the chair is positioned in its final location, you may now cut the shipping strap and remove the shipping screw from the reservoir cap.



## 8700DY CHAIR - QUICK INSTALLATION GUIDE

- 5** Lower backrest articulation bracket, and install the backrest frame using four ¼" bolts. NOTE: Use the chair functions to adjust the chair position to ease installation.

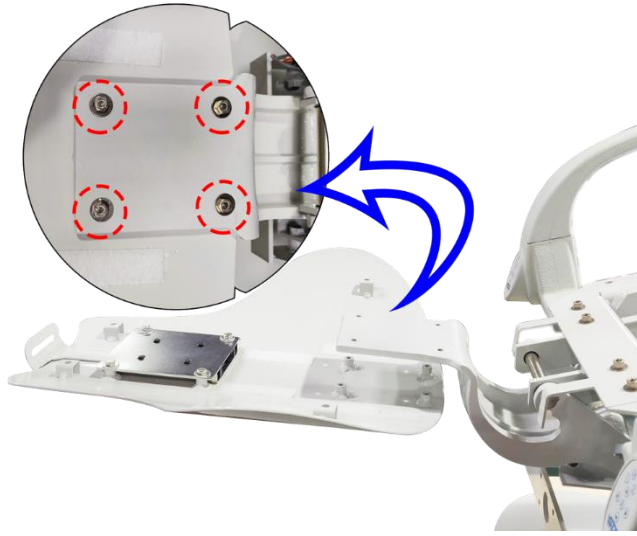

- 6** Put the headrest in position as shown in the image and install upholstery.



- 7** Re-install all covers and test all chair functions.

